

Instructions for subcutaneous self-injection

AMGEVITA® SureClick® disposable pre-filled pen

The SureClick® pre-filled pen allows you to self-administer injections of AMGEVITA® from the comfort of your home.

Important:

Always use this medicinal product exactly as your doctor or pharmacist has told you. Check with your doctor or pharmacist if you are not sure.

Prior to use, please note the following:

- ✓ Only administer the injection if your doctor or the practice team has shown you how to self-administer injections.
- ✓ Store the pre-filled pens in the refrigerator at 2 to 8°C.
- ✓ Do not use a pre-filled pen if it has fallen on a hard surface. Parts of the pre-filled pen may be damaged even if you cannot see any damages.
- ✓ Wash your hands thoroughly with soap and water.

Have the following ready:

- ✓ A new pre-filled pen
- ✓ Alcohol swabs
- ✓ Cotton ball or gauze
- ✓ Plaster
- ✓ Sharps disposal container


Step 1: Take an AMGEVITA® pre-filled pen from its packaging

Carefully lift the pre-filled pen straight up out of its packet approximately 15 to 30 minutes prior to injection so that it can reach room temperature. Place the original packaging with the non-used AMGEVITA® pens back in the refrigerator.

Never

- place the pre-filled pen back in the refrigerator if it is already at room temperature.
- attempt to heat up the pre-filled pen by using a source of heat, such as hot water or a microwave.
- shake the pre-filled pen.
- remove the yellow cap too early from the pre-filled pen.

Step 2: Check the AMGEVITA® pre-filled pen

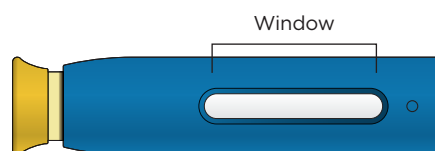


Make sure that the medicinal product in the window is clear and colourless to slightly yellowish. Check the expiry date.

Do not use the pre-filled pen if:


- the medicinal product is cloudy or discoloured or contains flakes or particles.
- parts appear to be cracked or broken.
- the pre-filled pen has fallen on a hard surface.
- the yellow cap is missing or is not securely attached.
- the expiry date printed on the label has been exceeded.

Always use a new pre-filled pen.

Prior to use

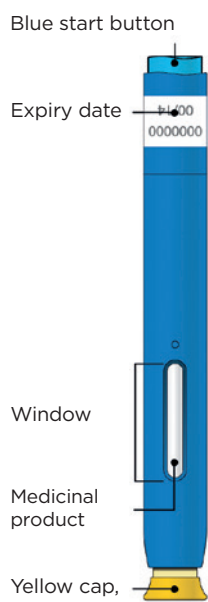


After use

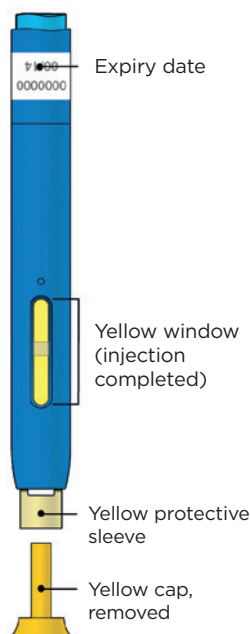

Important: The needle is inside the pen.

Step 3: Prepare and clean the site of injection

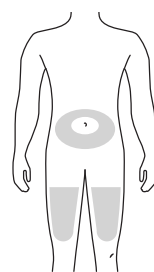

Choose the site of injection:


- Stomach (with the exception of an area of approx. 5 centimetres around the navel) or
- thigh

Disinfect the site of injection with an alcohol swab. As soon as the skin is dry, you are ready for the injection.

Important:

- Avoid touching the site of injection again prior to injection.
- Make sure that you do not use exactly the same site of injection that you used last time.
- Do not choose an area in which your skin is sensitive, damaged, red or hardened – avoid injecting in areas with scars or stretch marks.
- If you suffer from psoriasis, you should avoid injecting directly into raised, thick, red or flaky areas of skin or wounds.

Step 4: Pull the yellow cap straight off



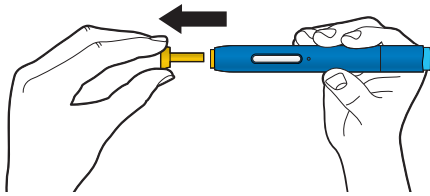


When you are ready for the injection, carefully pull the yellow protective needle cap straight off and away from the body. It is normal to see a drop of the medicinal product on the needle tip or on the yellow protective sleeve.

Never

- turn or bend the yellow cap.
- put the yellow cap back on the pre-filled pen.
- remove the yellow cap from the pre-filled pen before you are ready for the injection.

Step 8: Continue to press the pre-filled pen against the skin

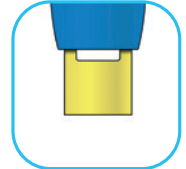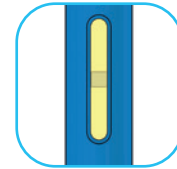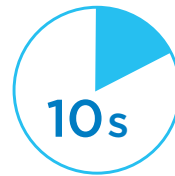


Slowly count to ten. The window turns yellow when the injection is complete. You may hear a second click. You can now remove the pre-filled pen from the skin; the needle is automatically covered.

Important: If you remove the pre-filled pen from the skin and the window has not turned yellow or it looks like the medicinal product is still being injected, this means that you have not received a complete dose. Call your doctor immediately.


Step 5: Create a firm surface for the injection

There are two methods of doing this:

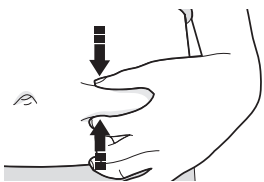

Squeezing method:

Squeeze the skin between your thumb and fingers firmly so that you create a surface of approximately 5 centimetres in width.

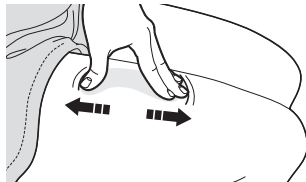

Stretching method:

Move your thumb and fingers firmly in opposite directions to create a surface of approximately 5 centimetres in width. You may hear a second click.

Important: Keep the skin stretched or squeezed together during the injection.


Step 9: After the injection



Immediately dispose of the used pre-filled pen in a sharps disposal container after use. Do not throw the used pre-filled pen and the sharps disposal container in the household waste and do not recycle them. Consult your doctor or pharmacist about their proper disposal.

Important: Do not reuse the used pre-filled pen. Always keep the sharps disposal container out of the sight and reach of children.

Step 6: Position the pre-filled pen

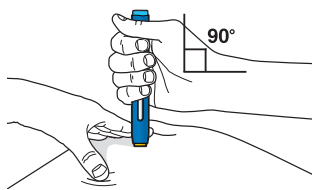


Keep the skin stretched or squeezed together during the injection. After removing the yellow cap, hold the pre-filled pen at a 90 degree angle to the skin.

Important: Do not touch the blue start button yet.

Step 10: Check the site of injection

If blood is visible at the site of injection, press down on the area for several seconds using a cotton ball or gauze. **Never** rub the site of injection. Attach a plaster or small dressing if necessary.

Step 7: Fix the pre-filled pen in position and press the blue start button

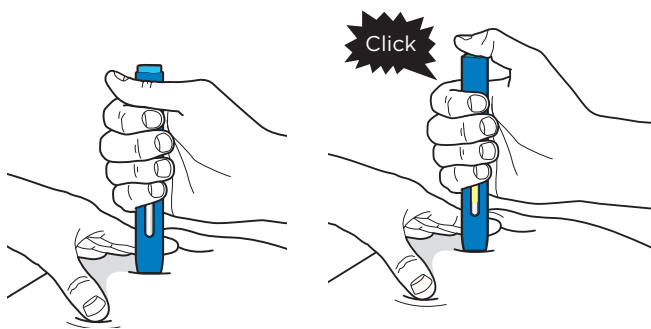

- Press the pre-filled pen firmly on your skin so that it does not move.
- When you are ready for the injection, press the blue start button. You will hear a click.
- The injection starts automatically.

AMGEN[®]
Care